
  
**S-CAPE PLUS**<sup>®</sup>  
THE EVACUATION SOLUTION

## Basic Model

**REF** 5N88888886 (In Blue Carrying Bag)

**REF** SCPLBP8990 (Basic Package)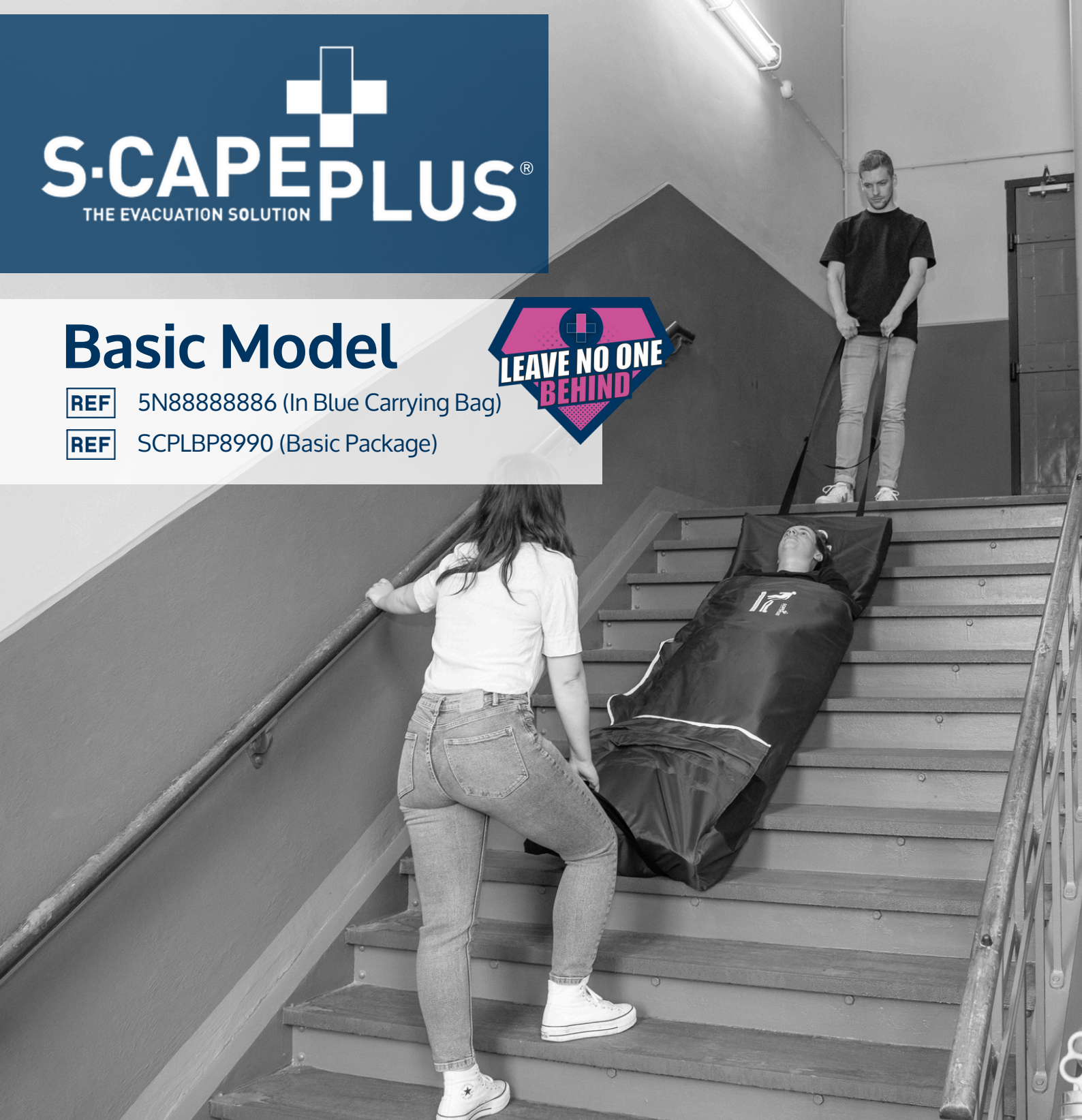

EN - Instructions For Use  
NL - Gebruiksaanwijzing  
DE - Gebrauchsanleitung  
LT - Naudojimo Instrukcijos  
HR - Rucno Koristenje  
FR - Instructions  
IT - Manuale Utente

NO - Manuell Styring  
SV - Manuell Hantering  
DAN - Manuel Håndtering  
FI - Manuaalinen Käsittely  
ESP - Manual del Usuario  
PL - Instrukcja Obsługi

# 1. General Information

## 1.1 Tetcon - Company Information

**Tetcon BV.**  
Tarasconweg 13  
5627 GB Eindhoven  
The Netherlands

Call: +31 40 298 99 55  
Email: info@tetcon-ge.com

VAT: NL854933530B01  
Chamber of Commerce: 62727869  
NATO NCAGE: H2PNI  
Websites: www.tetcon-ge.com | www.s-capeplus.com

Class 1 (EU 2017/745 (MDR))

### Customer outside the Netherlands?

Tetcon's Resellers are available in 50+ countries spread over six continents.

We can bring you in contact with a sales point near you.

## 1.2 Disclaimer

This manual contains information concerning the safe use of the S-CAPEPLUS. To ensure that the S-CAPEPLUS is used correctly and safely, this information must be read prior to placement and usage of the S-CAPEPLUS. The instructions are not all-inclusive.

Proper use of the S-CAPEPLUS requires training and periodic practice. Do not overestimate your skills, strengths, and abilities. Staircase evacuations can be difficult. You must be able to control the patients descent speed while sliding the patients down the staircase. In some instances (heavier patients, or a caregiver lacking training or sufficient strength) you should request the assistance of a second caregiver. Periodic practice using the S-CAPEPLUS should be an element of evacuation exercises. In all instances, compliance with the instructions is important. Only use at training or in case of emergency situations.

Retain this manual for future reference. Include it with the product in the event of transfer to users. The manufacturer and supplier cannot be held liable for any improper use!

The images and icons shown are for illustration purposes only and may not be an exact representation of the product.

## 1.3 Proprietary Notice

The information disclosed in this manual is the property of Tetcon BV., the Netherlands. Tetcon holds and reserves all intellectual property rights, proprietary design rights, manufacturing rights, reproduction use rights, and sales use rights thereto, and to any article disclosed therein except to the extent those rights are expressly granted to others or where not applicable to vendor proprietary parts.

## 1.4 Explanation of signs

### WARNING

This warning signal in the instruction manual is used to alert you to important information that must be followed to prevent accidents or damage of property. It is important to take note of all warning signals in this instruction booklet in order to ensure safe use of the product or service. Failure to heed a warning signal may lead to serious injury or damage to property.

### Note

This note signal in this instruction manual signifies a point that requires extra attention. Note signals are used when a task has a high risk of failure, causing hazard, or when it requires extra attention while performing a procedure.

### Tip

This tip signal alert you that a particular procedure of process should be performed with caution, or that certain safety guidelines should be followed.

# 2. Table of contents

	<b>General Information</b> 1.1 Tetcon - Company Information 1.2 Disclaimer 1.3 Proprietary Notice 1.4 Explanation of signs	<b>Page 2</b> Page 2 Page 2 Page 2 Page 2
	<b>Table of contents</b>	<b>Page 3</b>
	<b>Product Safety label</b> 3.1 Explanation product safety label 3.2 Example of the product label	<b>Page 4</b> Page 4 Page 4
	<b>Technical Specifications</b>	<b>Page 5</b>
	<b>S-CAPEPLUS Basic - Components</b>	<b>Page 6</b>
	<b>Safety and Maintenance</b> 6.1 Safety Information 6.2 Maintenance S-CAPEPLUS 6.3 Inspecting the S-CAPEPLUS 6.4 Inspecting Record	<b>Page 7</b> Page 7 Page 7 Page 7 Page 7
	<b>Storing and preparing for use</b> 7.1 Preparing before use 7.2 Storage options 7.3 Storing the S-CAPEPLUS Evacuation Mat 7.4 This is how you attach the Wall Cover to the wall	<b>Page 8</b> Page 8 Page 8 Page 8 + 9 Page 9
	<b>Instructions for use</b> 8.1 Instructions for use (EN, NL, DE) 8.2 Safety information (NL, DE) 8.3 Instructions for use (LT) 8.4 Safety information (LT) 8.5 Instructions for use (HR, FR, IT) 8.6 Safety information (HR, FR, IT) 8.7 Instruction for use (NO, SV, DAN) 8.8 Safety information (NO, SV, DAN) 8.9 Instruction for use (FI, ESP, PL) 8.10 Safety information (FI, ESP, PL)	<b>Page 10</b> Page 10 Page 11 Page 12 Page 12 Page 13 Page 14 Page 15 Page 16 Page 17 Page 18
	<b>Skills and Training</b> 9.1 Skills and Training 9.2 Training Record	<b>Page 19</b> Page 19 Page 19

# 3. Product safety label

## 3.1 Explanation product safety label

The product label is fixed to the S-CAPEPLUS.

Symbols appearing on product label:

 REF	Product Number	 LOT	Batch Number		Read User Instruction Manual Before Use
 Date of Manufacture	 Manufacturer	 KG	Weight	 MD	Medical Device
 CE	Conformité Européenne Mark		Laundry Instructions		Fire Retardant

## 3.2 Example of the product label

S-CAPEPLUS  
Cover (Basic Model)

REF

Idea manufactured in  
The Netherlands  
[www.s-capeplus.com](http://www.s-capeplus.com)

8 720726 730198

CE  MD

 100% PES/ANTIBAC

 yyyy-mm-dd

LOT xxxxxxxx

Tetcon B.V.  
Tarasconweg 13  
5627 GB Eindhoven  
The Netherlands

13.00cm

03.00cm

<b>Model name:</b>	S-CAPEPLUS Evacuation Mattress – Basic Model		
<b>Article number:</b>	<b>REF</b>	5N88888886	
<b>GMDN:</b>	66353		
<b>Basic UDI-DI:</b>	8720726730S-CAPEPLUSZT		
<b>GTIN:</b>	8720726730044		
<b>Classification fire retardancy:</b>	M1 / EN597		
<b>Anti bacterial treatment:</b>	Sanitized® TH 27-24		
<b>Size mattress (L x W x H):</b>	210 x 60 x 7 cm / 82.6" x 23.5" x 2.8"		
<b>Size packed in carrying bag:</b>	60 x 35 x 60 cm / 23.5" x 13.5" x 23.5" inches		
<b>Size packed in Wall Cover:</b>	100 x 60 x 17 cm / 39.3" x 23.5" x 6.69" inches		
<b>Weight:</b>		6,5 kg   14.5 lbs	
<b>Safe working load:</b>		200 kg   441 lbs	 <b>Do not exceed the safe working load!</b>
<b>Material:</b>	100% Polyester		
<b>Laundry instructions:</b>	<p>Clean by hand: Use a wipe and disinfectant</p> <p>In the washing machine: Remove the cover and place in the washing machine. You can wash the cover with the following washing instructions: Washing temperature, 40 degrees (Celsius), dry at a low temperature, do not iron, do not dry clean, do not bleach, do not use washing powder with fabric softener</p>		
<b>Warranty:</b>	S-CAPEPLUS has one-year guarantee, from the invoice date, on confection and manufacturing faults.		
<b>Declaration:</b>	<p>Produced in alliance with the European Regulations:</p> <p><b>CE MD</b></p> <p>The product is classified in Class 1 according to Regulation MDR 2017/745/EU. Applied standards: NEN-EN-ISO 13485:2016 EN NEN-EN-ISO 14971:2019 EN</p>		

# 5. S-CAPEPLUS Basic - Components

- 1 Pulling strap head end
- 2 Abdominal strap (Yellow)
- 3 Body Flap
- 4 Hook and loop fasteners
- 5 Feet flap
- 6 Pulling strap foot end
- 7 Fluorescent lining
- 8 Fire retardant and bacterial fabric
- 9 Foam mat
- 10 Zipper

# 6. Safety and Maintenance

## WARNING

These instructions contain information concerning the safe use of the S-CAPEPLUS. To ensure that the S-CAPEPLUS is used correctly and safely, this information must be read prior to placement and usage of the S-CAPEPLUS.

### 6.1 Safety Information

- Always observe the instructions given in the instruction manual when using the S-CAPEPLUS.
- Only use the S-CAPEPLUS in emergencies, such as a fire, power cut, gas leak, terrorist attack, natural disaster, extreme smoke or any other life-threatening situation or emergency.
- Product training with the S-CAPEPLUS must always be given by a certified emergency response team member.
- The use of alcohol and/or prescription and over-the-counter medicine may have a negative effect on the skill of the rescuer who pulls the S-CAPEPLUS.
- Do not remove any parts of the S-CAPEPLUS.
- Make sure the safety information and instruction manual are kept close to where the S-CAPEPLUS has been installed.
- This S-CAPEPLUS is not intended for recreational use.
- Do not use flammable vapours, materials of liquids, such as petrol, alcohol, etc., close to the S-CAPEPLUS.
- Not observing the instructions given in the instruction manual provided with the S-CAPEPLUS may result in serious physical injury or even death.
- Sharp edges or protrusions may damage the S-CAPEPLUS and, therefore, reduce the effectiveness of the product.
- Practice using the S-CAPEPLUS at least once a year in order to guarantee its correct and safe use in the event of an emergency.
- Contact the various rescue services (fire brigade, ambulance service, etc.) to inform them of the purchase and presence of the S-CAPEPLUS.
- The S-CAPEPLUS is not suitable to be pulled over a naked flame.
- The S-CAPEPLUS must be carefully checked for damage after it has been used. If damage is observed, the S-CAPEPLUS must not be reused. Furthermore, the S-CAPEPLUS must be folded up and put away each time after use.
- For hygiene reasons, the S-CAPEPLUS must be regularly cleaned. Always wash the S-CAPEPLUS separately and follow the cleaning and disinfection instructions.
- The S-CAPEPLUS has a 1-year warranty for material and manufacture faults, which is valid from the date of purchase. The manufacturer may ask you for proof of the purchase date. Therefore, it is recommended to keep the invoice in a safe place. Faults or problems with the use of the S-CAPEPLUS as a result of an accident, misuse, incorrect use, alterations, vandalism, incorrect placement, incorrect maintenance of administration, as well as overdue normal and routine maintenance, are not covered by the warranty.
- The manufacturer is not liable for (consequential) damage as a result of the use of the S-CAPEPLUS.

**Never overestimate yourself! Evacuation via a staircase can be very dangerous! You will have to support a heavy weight and you must be certain that you are capable and strong enough to keep this weight under control all the way down the staircase. If in doubt as to whether you are able to evacuate somebody down a staircase yourself, you must always ask a second care worker to help you or leave the task to someone else. Your own safety is of the utmost importance! Regularly practicing using the S-CAPEPLUS during evacuation exercises and always observe the instructions given in this manual.**

### 6.2 Maintenance S-CAPEPLUS

Maintaining and cleaning the S-CAPEPLUS evacuation mattress is important to ensure efficient operation in the event of an emergency. It also ensures that your evacuation mattress is up-to-date and properly maintained. Set up and follow a maintenance schedule. The schedule below represents minimum maintenance and you should modify your own program based on your needs.

As well as inspecting the S-CAPEPLUS after each use and before each use, it must be inspected by a competent person. Records of inspecting can be recorded in the 'Inspection & Maintenance Manual' below.

Procedure	Frequency
Disinfection	After use or as needed.
Cleaning/washing	Cleaning and washing after each use. Also as needed.
Inspecting	Before installation and after each use.

#### Note

Check page 5 for laundry instructions

### 6.3 Inspecting the S-CAPEPLUS

You should always inspect the S-CAPEPLUS before installation and after each use. This helps to ensure that it is in good condition and will not cause any problems during evacuation.

Check for signs of any damages that could weaken the S-CAPEPLUS such as broken crossboxes, broken stitching, holes, etc.

The inspecting record on the right you can use to check if the S-CAPEPLUS is still in good condition and can be used. It outlines all the components of the S-CAPEPLUS and provides information on checking the condition of each part. It shows where to look for possible problems, specific things to check, and general signs of wear and tear. By using this diagram, you can quickly and easily determine if the S-CAPEPLUS is still in good condition and if it can safely be used.

### 6.4 Inspecting Record - before using and after each use

- Are all components present? (Refer to page 6., S-CAPEPLUS Basic - Components)
- Are the flaps in tact? (Refer to page 6., S-CAPEPLUS Basic -Components)
- Is all webbing in good condition with no cuts or frayed edges?
- Is webbing stitching in good condition with no frayed or broken threads?
- Are body flaps with hook and loop fasteners operational and do they attach and detach smoothly? (Refer to page 6., S-CAPEPLUS Basic - Components)
- Is the fabric intact and not excessively worn?

**If inspection shows damage or excessive wear, remove the S-CAPEPLUS from service for further inspection or retirement.**

**Do not attempt to repair or replace any materials or components.**

# 7. Storing and preparing for use

## 7.1 Preparing before use

Always make sure you have read the product manual first. The advice is to train every six months with people who have basic knowledge of emergency evacuation.

- **Stay with patient at all times**

Never leave the patient unattended. An unattended patient can be injured.

- **Use appropriate techniques and procedures**

Refer to local protocols and guidelines regarding manual handling of persons or objects.

- **Using the S-CAPEPLUS requires one or two trained operators**

Trained personnel will be able to perform the task on their own. However, in most situations, operators will want or need additional help.

### WARNING

Improper installation can result into injury or malfunction.

### Note

Failure to properly install the S-CAPEPLUS can cause you to lose valuable time during the emergency evacuation.

## 7.2 Storage options

Wall Cover Green  
Photo 1

Portable Storage Bag  
Photo 2

Steel Storage Cabinet  
Photo 3

## 7.3. Storing the S-CAPEPLUS Evacuation Mat

Steps explained:

- Lay the S-CAPEPLUS on the ground and make sure that the entire mattress is exposed.
- Close the body flaps but make sure the hook and loop fasteners are not entangled to each other. This saves time when unfolding the evacuation mat.
- Fold the S-CAPEPLUS in half (for Wall Cover or Storage Cabinet - see photo 1).
- Fold the S-CAPEPLUS for the blue storage bag (see photo 2). Option 1: Place the S-CAPEPLUS on a flat surface. Make sure that the head icon is facing upwards. Fold the mattress from both head and feet end inwards to the middle of the mattress. Now fold once more. Put your own weight on the mattress so that the folded mattress remains together. Place the S-CAPEPLUS inside the blue storage bag. Option 2: Place the mattress on a flat surface. Make sure that the head icon is facing upwards. Now start from the head end and fold the mattress three times. Put your own weight on the mattress so that the folded mattress remains together. Place the S-CAPEPLUS inside the blue storage bag (page 9).
- Store the evacuation mat in the storage option that you purchased.
- Assess and determine the best place to install the S-CAPEPLUS Evacuation Mattress. Preferably mount it to the wall (7.4).
- Store the S-CAPEPLUS in its storage cover away from direct sunlight and protected from dust, moisture and corrosive environments.

Photo 1

Scan QR code to see on video how to place a  
Evacuation Mat in the storage cabinet

### 7.3. Storing the S-CAPEPLUS Evacuation Mat

Photo 2 - option 1

Photo 2 - option 2

### 7.4 This is how you attach the Wall Cover to the wall

What do you need:

- Bag with screws (included in the package)
- Drill (not included in the package)
- Tape measure (not included in the package)
- Screwdriver (not included in the package)
- Pencil (not included in the package)
- Spirit level (not included in the package)


Scan QR Code to see on video how to attach the Wall Cover to the wall.

#### Optional and recommended:


Photoluminescent Sign (20x20 cm) to highlight the location of the S-CAPEPLUS Evacuation Mattress. It can be placed above all storage options (7.2) to increase and promote visibility of the emergency evacuation device.

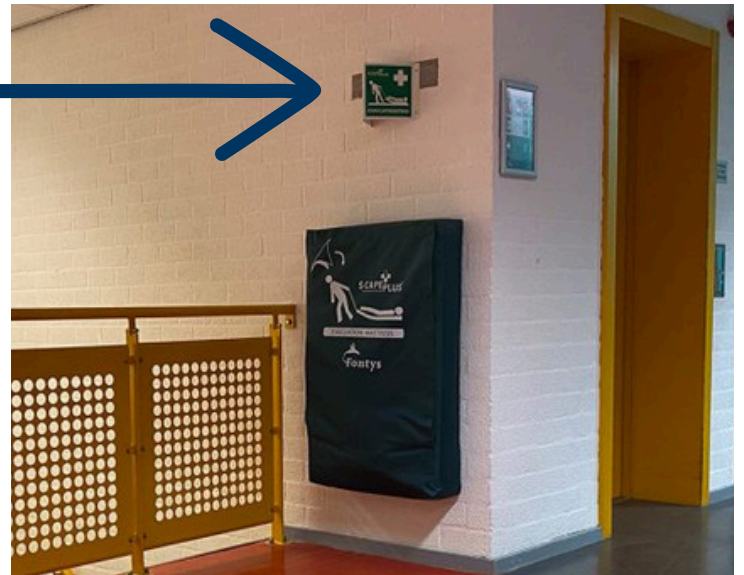Please read these instructions before use (IFU). Do not discard: keep for future reference.

1

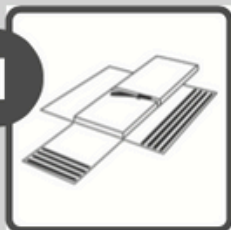
**ENG** Remove the S-CAPEPLUS from the bag. Fold out the flaps

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2

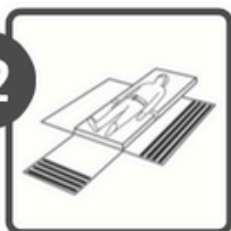**ENG** Position the patient in the middle of the matress. Make sure the positioning belt is placed right around the person's torso by pulling the belt through the ring on the right hand side. Fix this in place with the hook and loop fasteners. Position the arms along the body and above the belt.

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Fixate the abdominal strap as tight as possible.

3

**ENG** Now firmly pull the long flaps towards each other. Connect and stick the hook and loop fasteners to the right flap with the hook and loop fasteners on the left flap. Close this up as tightly as possible. Now close up the flap at the foot end. Again, do this as tightly as possible.

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Personnel using the S-CAPEPLUS in emergency situations are responsible for knowing the techniques involved.

4

**ENG** Take a firm hold of the loop at the foot end, allowing you to evacuate the patient in a dragging motion. A second rescuer can use the strap at head end to navigate through small hallways and make quick turns. The person at the head end can also lift up the matress slightly to provide support to the victim's head and neck.

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Depending on ground conditions, the friction generated during dragging may abrade or damage the S-CAPEPLUS fabric.

5

**ENG** If you find yourself on a higher floor, you will need to use the stairs to make your way downstairs, as you won't be able to use any possible lift present during a fire. This will not present any problems, providing you take command of the stairs. Make sure you have enlisted the help of a second rescuer to hold the loop at the head end. This person can be responsible for any necessary braking when making your way down the stairs. Position yourself a few treads lower and pull the patient towards you. Carefully lower the matress once the patient has crossed the stairwell with his/her buttocks. Make sure you keep a firm hold of the railing at all times and always maintain contact with the S-CAPE PLUS. Don't miss out any treads when going down the stairs and gradually do so step by step.

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SCAN ME

Make sure you keep a firm hold of the railing at all times and always maintain contact with the S-CAPEPLUS. Don't miss out any step when going down the stairs and gradually do so step by step.

## English Safety Information on Page 7

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---

## A few words from our founders

Thank you for joining 'The Leave No One Behind' Movement. Each and every piece of evacuation equipment is based on our core design values: 'simplicity, speed and comfort'. These values will help you to become an expert with the range of evacuation equipment. Every product serves Tetcon's mission to Leave No One Behind. That means that we want to bring safety and security enthusiasts together; to inspire, strengthen and promote that vulnerable people are not left behind in an emergency event.

Good Luck and have a safe day.

The Hezemans Family

---

# S-CAPE PLUS<sup>®</sup>

THE EVACUATION SOLUTION

Tetcon Global BV | Tarasconweg 13 | 5627 GB Eindhoven | The Netherlands  
info@tetcon-ge.com | +31 40 298 99 55 | www.s-capeplus.com | www.tetcon-ge.com